Clinisupplies

 HR HealthCare

Sole distributor for Clinisupplies Limited
in United States of America

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EN: Instructions for Use

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Distributed in the USA by: HR HealthCare | 2600 Eastern Blvd., Ste. 201 York, PA 17402 USA
Tel. 800-985-9048 | customer@hrhealthcare.com | hrhealthcare.com

Vitalcare (Nanjing) Co., Ltd.,
No. 86-8, Shuanggao Road, Gaochun District, Nanjing, 211316 Jiangsu, P.R. China

hrhealthcare.com/symbolsglossary

HR HealthCare and the HR HealthCare logo are trademarks of
HR HealthCare. ©2025 HR HealthCare. All rights reserved.


Clinisupplies and Aquaflush logo are trademarks of
Clinisupplies Limited - All rights reserved

CSF01139_AF_USA_1a_v2
LBL-0050 Rev. 3 (07/2025)

REF AFLS, AFAS, AFLM, AFAM,

Thank you for choosing Aquaflush®. Aquaflush® Transanal Irrigation (TAI) System is designed to be a simple and effective way for you to control your bowel management routine.

Please read this document carefully, including the warnings and precautions. If you have any questions, please contact us at HR HealthCare or discuss any concerns with your healthcare professional.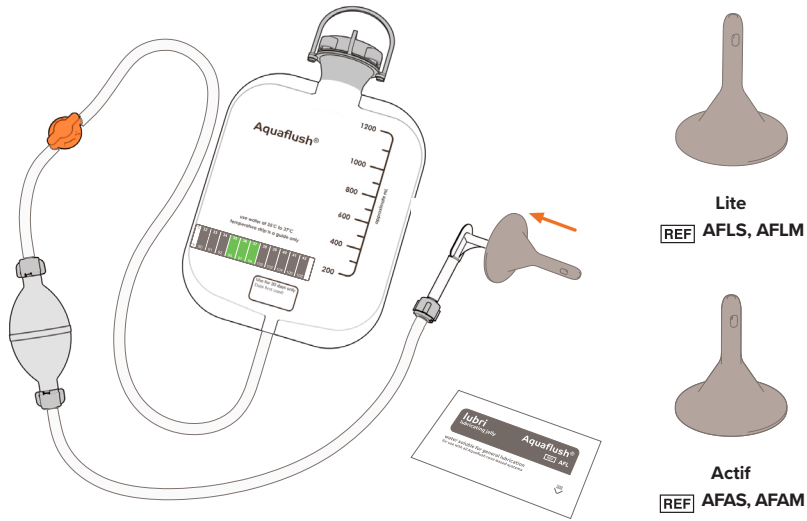

INDICATIONS FOR USE

Aquaflush® Transanal Irrigation System must only be used by the person that it is prescribed for following a thorough assessment by a healthcare professional.

INTENDED USE

This device is intended to be used by people with chronic constipation, chronic fecal incontinence, incomplete emptying, or neurological bowel dysfunction.

CONTRAINDICATIONS

Caution: The following list does not rule out transanal irrigation as a bowel management program, but you should have consulted a healthcare professional to ensure that the system is right for you. Children under 5 must be reviewed by a pediatric specialist before starting transanal irrigation.

- Inflammatory bowel disease (e.g. Crohn's disease or Ulcerative Colitis and Ischaemic Colitis)
- Rectal pain or bleeding (e.g., anal fissure, severe hemorrhoids)
- Diverticular Disease
- Autonomic Dysreflexia
- Recent spinal cord shock following spinal trauma
- Radiation therapy in the abdominal or pelvic region
- Abdominal or anal surgery
- Recent colonic biopsy or endoscopic polypectomy
- Cancer in the abdominal or pelvic region
- Long term steroid therapy, Anticoagulant Therapy
- Pregnancy
- Nursing mothers

ADVERSE EVENTS

Any serious incident that has occurred with this device should be reported to the manufacturer and competent authority.

WARNINGS AND PRECAUTIONS

You must consult a healthcare professional if any of the following occur:

- Blood in the feces, weight loss or abdominal pain
- Significant changes in the frequency, colour and consistency of the stool


Rare occurrence: Bowel perforation is extremely rare but is serious and life threatening if it occurs and will require urgent attention. If you experience any of the following symptoms during or after transanal irrigation, contact your healthcare professional immediately:

- Severe and sustained abdominal pain or back pain, especially if combined with fever
- Sustained bleeding

STEP-BY-STEP INSTRUCTIONS

1. Before you start: Wash your hands thoroughly and ensure you have everything you need close by. Start by ensuring the orange tap on the tube is in the horizontal closed position as shown in the diagram below and the silicone cone has been attached to the 'elbow' connector. (Figure 1)


Figure 1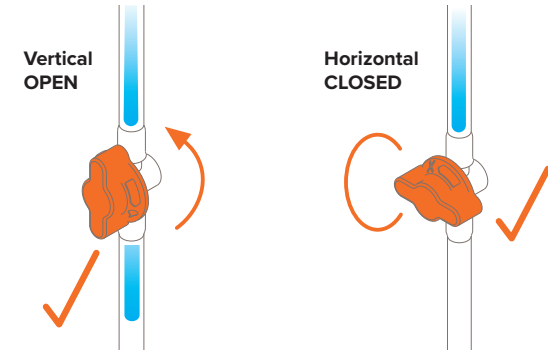

2. Preparing your system for use: Unscrew the waterbag cap and fill the bag with water at a temperature between 35°C to 37°C (95°F to 98°F), which is close to body temperature. (Do not use cold or hot water). Fold the bag at the water line to remove any trapped air before securely closing the cap. This process ensures that no air is trapped in the bag, preventing the risk of pumping air into the system. It is important you do not pump air into the bowel. (Figure 2)


User Tip: The temperature gauge on the bag will indicate when the water in the correct range.

Figure 2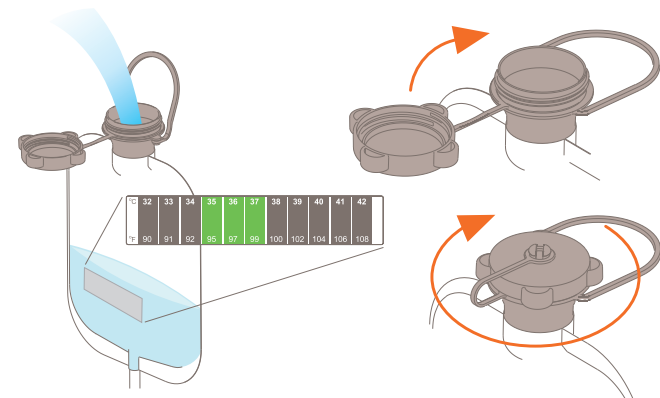


3. Place the waterbag on a flat surface or on the floor using the Aquaflush® Bag Stand provided or hang it above shoulder height using the built-in hanger with an 'S' hook. The hanging option uses gravity, allowing the system to free flow without the need for a pump and should be hung above the height of the toilet to avoid getting air into the system. You can only free flow if the water bag is above the height of the person doing the irrigation. (Figure 3)

Figure 3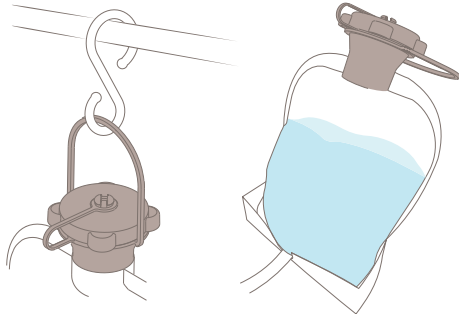

4. Removing air from the system: Hold the cone over the sink, open the orange tap into the vertical open position (see diagram) and allow the water to pass through the tubing and into the pump. Squeeze the pump a few times to clear any air in the system before use. Turn the orange tap back into the horizontal closed position. Now your Aquaflush® System is ready for use. (Figure 4)


Figure 4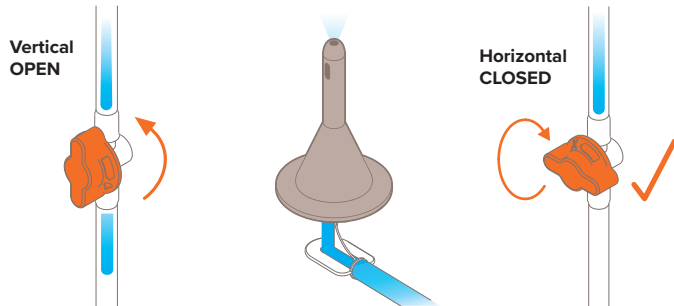

5. Apply the lubricant provided to the tip and shaft of the cone and the anus if required. Sit on the toilet and gently push the cone into the rectum at the correct angle of insertion advised by your healthcare professional. Turn the orange tap to the vertical open position fully. (Figure 5)


User Tip: If you have difficulty correctly inserting the cone, consult your healthcare professional.

Figure 5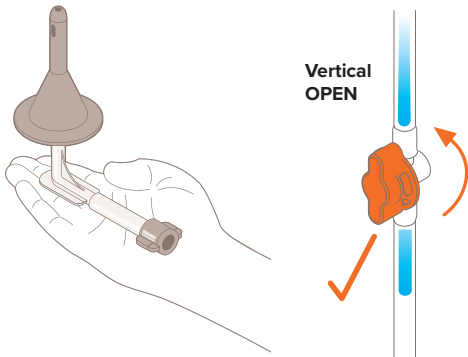


6. Gently pump the volume of water advised by your healthcare professional into the rectum, monitoring the volume scale on the water bag (usually between 300ml to 500ml). (If any discomfort is felt stop pump immediately, close the orange tap to the horizontal position and seek advice from your healthcare professional). When you have finished pumping, turn the orange tap to the horizontal closed position and remove the cone from the rectum. Remain seated on the toilet. Your bowel may take up to 30 minutes to empty, and it is recommended to wait an additional 5 minutes after everything appears to have been expelled. (Figure 6)

Figure 6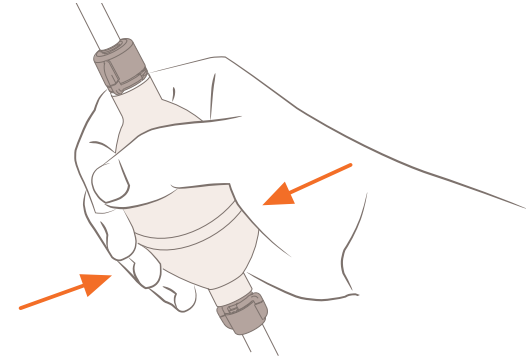

7. Using the disposal bag provided, remove the cone from the 'elbow' connector and dispose according to your local authority guidelines. The cone is intended to be used once. Wash the water bag, tubing and pump with mild soapy water and hang to dry ensuring the orange tap is in the vertical open position and the pump is disconnected from the water bag. (Figure 7)

Figure 7

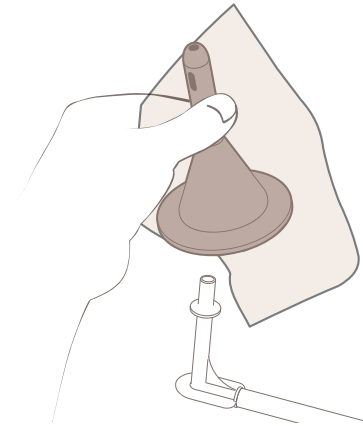STORAGE AND HANDLING

Cones are single-use, the rest of the system can be used for 30 days.

SAFETY: Ensure you store your Aquaflush® System away from the reach of small children to avoid risk of strangulation from tubing, suffocation from plastic bags or choking in the event of small components coming loose from the product.

Do not use after the date of expiry, which can be found on the carton label.

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